[english version below]

smartALF Release Notes

Änderungen in der Version 1.4.0.2 (07/2026)

Softwarestart (neu)

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Changes in version 1.4.0.2 (2026/07)

Software Startup (new)

- When the software starts, a window opens displaying status messages.
- If no smartALF hardware is detected, the search can be restarted or the software can be launched in demo mode.

Demo Mode (new)

- The smartALF software can be started in demo mode without connected hardware.
- In demo mode, all functions are available as usual; however, no audio signals are played.

Customer View (new)

- The customer view displays the current status of the ALF session in a web browser.
- During signal playback, the frequency being played on each side is displayed live.
Clicking the "Connect" button establishes the connection, allowing the customer to connect via the displayed QR code. Clicking the QR code also opens the browser.
- The device only requires an internet connection.
- **Guest Access vs. Permanent Access**
 - Guest access is intended for customers' devices and is only valid for the current ALF session.
 - Permanent access is intended for audiologists' devices and remains valid for a configurable period (e.g., one day).
 - If the permanent access code is accidentally shared with customers, it can be invalidated in the settings and a new permanent access code can be generated.

Settings Window (new)

- All configuration options are consolidated in a single settings window.
Customer Language (new):
 - In addition to the user interface language, a customer language can be selected.
 - This determines the language of the audiobooks being played.

Additional Changes

- Integrated player with drag-and-drop navigation for skipping forward and backward within audiobooks (Steps 2 & 6).
- The number of signal repetitions can only be changed outside the settings when Expert Mode is enabled (Steps 4 & 5).
- The report has been visually improved.
- English audiobooks have been added.

Changes in version 1.3.6.6 (02/2026)

Progress Bar

- A new progress bar now continuously displays your current status within the fitting process. Using the progress bar, you can move to the next step (provided the current step has been completed) as well as return to all previous steps.
- An **expert mode** can optionally be activated in the general settings. When Expert Mode is enabled, you can navigate directly to all steps using the progress bar. Enabling/disabling expert mode will be saved for the next software start.

OnTop Mode

- After completing the hearing aid selection, a new OnTop Mode becomes available. In this mode, the smartALF application is displayed in a small window that remains in the foreground.
- All steps can be fully completed in OnTop Mode; skipping steps is not possible.
- You can switch between the normal mode and OnTop Mode at any time, regardless of the current step.

Feedback

- Via the Help tab, you can now provide direct feedback on the smartALF application.
- Additionally, you can submit new hearing aid models to us so that we can integrate them into smartALF.

Further Changes

- The hearing aid selection panel can now be collapsed after startup to save screen space.
- Tablet scrolling has been improved and is now possible by pressing and dragging.
- Notification buttons have been enlarged to improve usability on tablets.
- Anchor frequencies can be changed without error: The user is asked whether data should be saved in the report (if available) and then the measurement is restarted.
- It is now possible to switch back and forth between the two frequency band modes; all adjusted frequencies are saved independently and also displayed in the report.
- During the control step (step 6), the audiobook/volume combinations are saved when the checkbox is clicked, and all clicked combinations are also displayed in the report.
- In steps 4 and 5, it is now possible to select the signal playback frequency, rather than the playback duration. The playback duration can still be changed in the general settings.
- Hearing aid models are no longer sorted alphabetically, but according to HearingAids.ini.
- Spanish translations have been improved.